

Pressure Gauge Check to be completed prior to use

BOC 400CD INHALO[®] Medical Oxygen Cylinder



- The position of the pointer is upwards.
- Read the cylinder pressure.

To check that the pressure gauge is working properly, lay the cylinder down from a vertical position to a horizontal position. Watch the position of the pressure gauge pointer. It should not move.

The 400CD INHALO[®] and Pressure Gauge in a **Vertical** Position.



The 400CD INHALO[®] and Pressure Gauge in a **Horizontal** Position.

- The pointer reads the same cylinder pressure, the gauge is working as expected, indicating the gas contents of the cylinder. Please continue to use the 400CD cylinder.



The 400CD INHALO[®] and Pressure Gauge in a **Horizontal** Position.

- The position of the pointer is still upwards, the cylinder pressure reading has changed, the pressure gauge is **FAULTY**.

If your pressure gauge is faulty please contact BOC to arrange a free of charge Cylinder Replacement on 1300 363 109.